




NucAP™ Pyoderma Tri-plate (Clind/Ox) Interpretation Guide

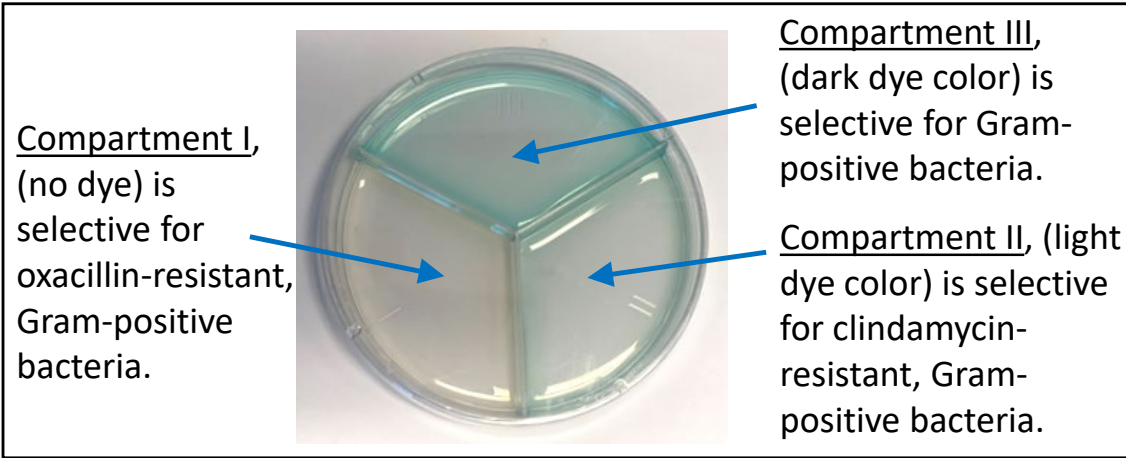


Note that orientation of compartments is different in upper image because plates are flipped below.

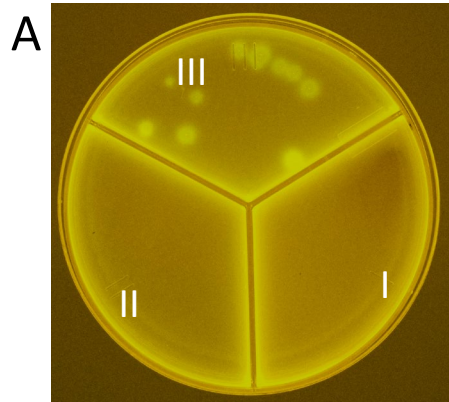


Oxacillin-Sensitive & Clindamycin-Sensitive Gram-Positive Bacteria

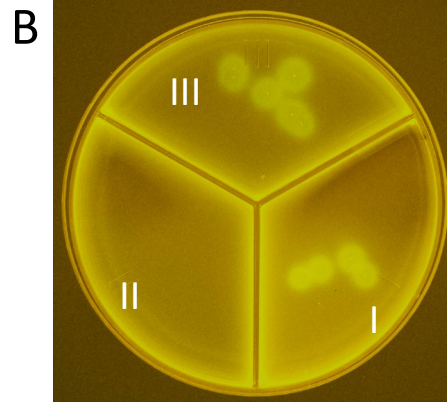

Oxacillin-Resistant & Clindamycin-Sensitive Gram-Positive Bacteria

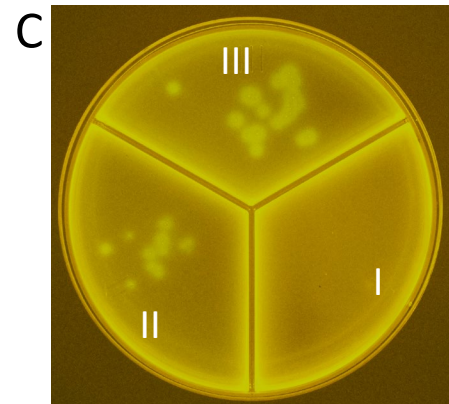

Oxacillin-Sensitive & Clindamycin-Resistant Gram-Positive Bacteria

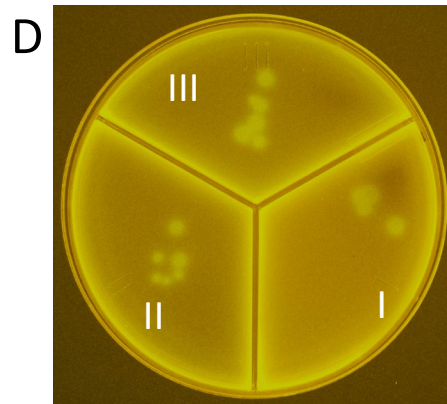

Oxacillin-Resistant & Clindamycin-Resistant Gram-Positive Bacteria

These are the 4 fluorescence patterns typically seen for positive samples after incubation at 37 °C for 8 hours. Users are directed to the Product Insert for a complete product description.